

Universally popular vertical fold flat FFP3 disposable respirator. Packed individually for easy hygienic dispensing. Space saving design allows the mask to be stored whilst not in use. Integrated exhalation valve reduces breathing resistance and heat build up for superior user comfort.

Light and comfortable with a soft foam nose seal and adjustable nose clip that moulds to the users face to improve face fit percentage and enhance compatibility with eyewear and reduced fogging.

Colour coded nose clip, valve and strap for easy identification in the workplace.

Maintenance free.



## FEATURES AND TECHNOLOGY



**INDIVIDUALLY BAGGED**



**REDUCED HEAT BUILD UP**



**INDIVIDUALLY PACKED**



**SINGLE USE**



**LOW BREATHING RESISTANCE**



**PROTECTION LEVEL**



## CERTIFICATION AND STANDARDS

**EN 149: 2001+A1: 2009 - FFP3 NR** This mask meets the requirements of EN149:2001 FFP3 and offers protection against non-toxic and low-to-average toxicity and high toxicity solid and liquid aerosols (e.g. oilmists) in concentrations up to 50 x MAC/OEL/TLV or 20 x APF

**STORAGE / TRANSPORT** Keep away from direct sunlight; store in a cool dry place. Keep away from ozone sources or naked flame. Store in their original packaging. Store between 0°C and 40°C.

**PRECAUTIONS BEFORE USE** A properly selected respirator is essential to protect your health. Before using this respirator consult a suitably qualified Safety Professional to determine the suitability of the product for your intended use.

**OBsolescence** When stored as recommended, these masks will not suffer a change in mechanical properties for to up 4 years from date of manufacture shown. Once this date has been reached, this product should be withdrawn from use.

**TRANSPORTATION** During transportation, ensure the product is well packaged to prevent damage that may impair the masks efficiency.

**CAUTIONS AND LIMITATIONS** 1. Not for use in atmospheres with less than 19.5% oxygen. 2. Do not use in atmospheres immediately dangerous to life or health. 3. Do not exceed maximum use concentrations established by regulatory standards 4. Failure to properly use and maintain this product could result in injury or death 5. All approved respirators should be selected, fitted, used and maintained in accordance with MSHA, OSHA and other applicable regulations. 6. Never substitute, modify, add or omit parts in the configuration as specified by the manufacturer. 7. See information on this packaging for use and maintenance of these respirators. 8. NIOSH does not evaluate respirators as surgical masks. 9. Fit according to supplied instructions.



**PACK of 10**